

# Use Instructions

## Anterior Holder

WINGERS-A (ANTERIOR, PURPLE, 088-028 SM./088-030 LG.)

### To Use With Hemostat:

**1. To Apply:** Slide sensor into holder pocket, exposure slide (Fig.1) facing bite bars located on one side of holder.

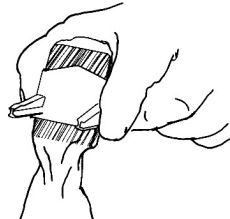


Fig.1

**2. Hemostat Use:** A hemostat can be clipped into the channel (Fig.2) provided on either "bite bar" for in mouth positioning.

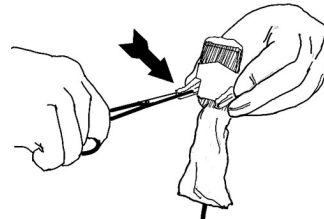


Fig.2

**3. Placement:** Sensor can now be placed in a multitude of positions in the mouth. Ample room is provided between bite bars on the holder (Fig.3) for placement of Endodontic files or rubber dam.

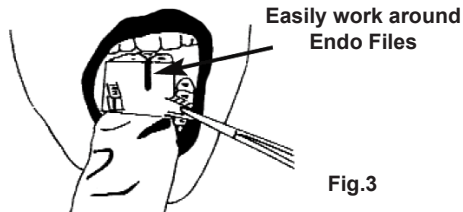


Fig.3

### To Use With Anterior Bar & Aimer Ring:

ANTERIOR BAR (PURPLE, 088-040) AIMER RING (WHITE, 088-044)

**1. Place Sensor In Holder:** Place sensor in holder, sensor should be centered in holder. Use center line on holder to help center sensor (Fig.4).

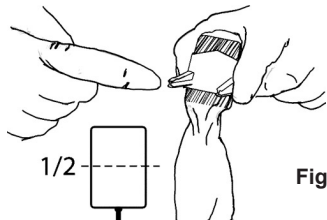


Fig.4

**2. Apply to Bar:** Slide Anterior holder into the pronged teeth at the "TOP" of the Anterior Bar (Fig.5) (accommodates small and large holders)

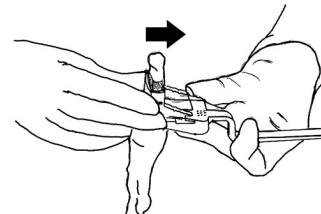


Fig.5

**3. Aimer Ring Placement:** Slide Aimer Ring onto bar. Place sensor wire in the either hook located at base of Aimer Ring (Fig.6).

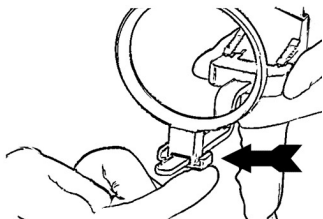


Fig.6

**4. Placement:** Place sensor behind anterior teeth, patient should bite on "pebbled bite surface" provided on bar. Depending upon patient bite, a cotton roll may be needed for proper positioning. To obtain images of the lower anterior teeth, inverter bar and ring (Fig.7).

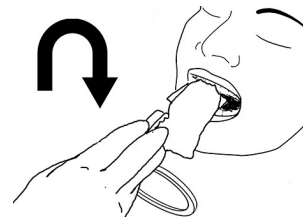


Fig.7

**AIMER RING AND BARS ARE STERILIZABLE:** Use cold sterilants, rinse thoroughly. Heat sterilize Maximum 105 °C. **Wingers Holders are disposable, single use only. WARNING! WINGERS HOLDERS ARE NOT DESIGNED FOR STERILIZATION. HEAT WILL MELT THE HOLDERS, LIQUID STERILANTS MAY BE ABSORBED BY THE HOLDERS. MANY LIQUID STERILANTS ARE TOXIC AND CORROSIVE TO HUMAN TISSUE!**