


Use Instructions


Bitewings

WINGERS-H (HORIZONTAL, PINK, 088-026 SM./088-024 LG.)
WINGERS-V (VERTICAL, GREEN, 088-036 SM./088-038 LG.)


1. To Apply: Slide a sensor into either bitewing and center (Fig.1), exposure side of sensor facing arrowed edges on holders. User will need to experiment with the Vertical bitewing holder regarding cable in the "up" or "down" position as image will vary with different types of sensors.

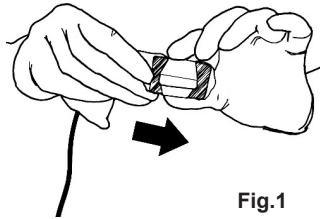

2. Placement: Place in patients month mid tongual (Fig.2) with bitewing horizontal to the occlusal plane. Patient should bite on wing.

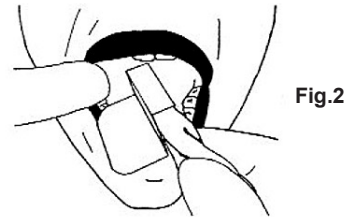

***TIP:** If Patient bites on cable, move cable to an area of the mouth (Fig.3) where it can easily exit the teeth.

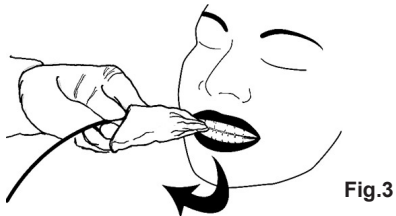

AIMER RING AND BARS ARE STERILIZABLE: Use cold sterliants, rinse thoroughly. Heat sterilize Maximum 105 °C. **Wingers Holders are disposable, single use only. WARNING! WINGERS HOLDERS ARE NOT DESIGNED FOR STERILIZATION. HEAT WILL MELT THE HOLDERS, LIQUID STERILANTS MAY BE ABSORBED BY THE HOLDERS. MANY LIQUID STERILANTS ARE TOXIC AND CORROSIVE TO HUMAN TISSUE!**